OM-H0190E 002

OPERATION MANUAL

External Coolant Straight Handpiece

Ti-Max X

X-SG65L / X-SG65

MADE IN JAPAN

Intended Use

This handpiece is designed for clinical dental use and implant surgery.

CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the handpiece always consider the safety of the patient.
- Do not attempt to disassemble the handpiece nor tamper with the mechanism.
- Check the vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using the handpiece immediately and contact your dealer.
- Should the handpiece function abnormally, cease operation immediately and return the handpiece to the dealer for repair.
- Do not allow any impact on the sheath. Do not drop the handpiece.
- Always keep bur shanks clean. Dirt or debris in a chuck could cause poor bur concentricity or low chuck retention force.

1. Connection / Disconnecting the Handpiece and Motor

(1) Connecting


Push the handpiece on to the motor taking care to keep it in alignment.

X-SG65L Only

Twist the handpiece until it locks into position with a clicking sound.

(2) Disconnecting

Hold the motor and the handpiece, and pull it out.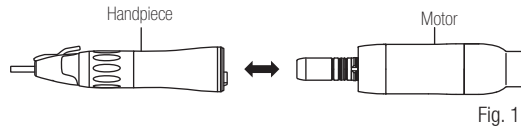

2. Mounting and Removing Bur


(1) Handpiece Bur

Inserting the Bur

1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.

2) Insert the bur.

3) Turn the Bur Lock Ring in the direction of the mark [C] until it clicks. Make sure Black Dots align (Fig. 2).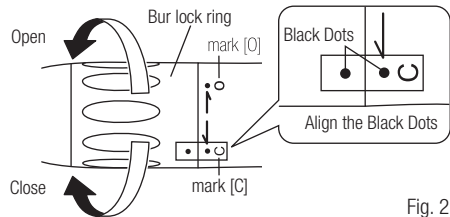

Removing the Bur

Turn the Bur Lock Ring in the direction of mark [O] until it clicks and remove the bur.

(2) Contra-angle bur

Inserting the Bur

1) Turn the bur lock ring in the direction of mark [O] until it clicks.

2) Insert the supplied Bur Stopper into the chuck. (Fig. 3)

3) Insert the Contra-Angle Bur until it stopped.

4) The bur will be chucked by turning the bur lock ring in the direction of mark [C] until it clicks and Black Dots line up.



Removing the Bur

Turn the Bur Lock Ring in the direction of mark [O] until it clicks and remove the bur.

Remove the Bur Stopper

1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.

2) Insert the supplied Bur Stopper puller into the chuck. Pull out the Bur Stopper by hooking at the slit of Bur Stopper. (Fig. 4)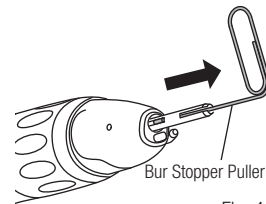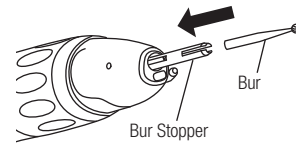

CAUTION for Bur Usage

Heat Generation Problem

- Bur position is improper or it is not mounted.
--> Mount the bur correctly.
- During operation, Bur Lock Ring had been turned.
--> Do not turn the Bur Lock Ring while the handpiece is rotating.

Bur Jump Out Problem

- Incorrect mount of the bur/bur stopper.
--> Mount the bur/bur stopper firmly.
- During operation, Bur Lock Ring had been turned.
--> Do not turn the Bur Lock Ring while the handpiece is rotating.

- Mount short shank contra-angle bur.
--> Do not mount this bur.

Bur Breakage Problem

- Mount damage or sub-standard burs.
--> Do not mount those burs.
- Exceed drive motor speed or manufacture's recommended bur speed.
--> Do not exceed drive motor speed or manufacture's recommended bur speed.

3. External Irrigation Nozzle

Connect the irrigation tube to the external irrigation nozzle firmly.

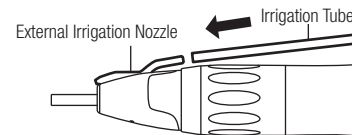
Fig. 5

4. Maintenance

CAUTION

After each patient, be sure to keep the handpiece after cleaning, lubrication, and sterilization. If the blood is left on the handpiece, it can be clotted causing rusty.

Refer to the Maintenance Guide **OM-Z0135E**.

This handpieces can be autoclavable up to Max.135°C.

This handpieces can be cleaned and disinfected with a Thermo-Disinfector.

*Cleaning

Clean the external irrigation nozzle inside with clean water.

5. Cleaning the Glass Rod Optic

Clean the Glass Rod Optic with cotton swab soaked in alcohol.

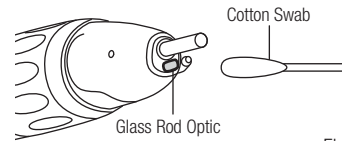
Fig. 6

CAUTION

Do not use a sharp tool to clean the glass rod optic. It could damage the glass rod optic and decrease the light transmission. If it becomes dim, contact your dealer.

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ENGLISH Specification

Model	X-SG65L	X-SG65
Gear Ratio	1:1	
Allowable Max Speed	40,000 min ⁻¹ (rpm)	
Bur	ø2.35mm Handpiece Bur and ø2.35mm CA Bur	
Bur applied Length	29.4mm (Handpiece Bur)	
Max Bur Length	44.5mm (Handpiece Bur)	
Max Bur diameter	Ø4.0	
Irrigation	External Irrigation	
Optic	Glass Rod	-
Use Environment	Temperature : 0 – 40 °C, Humidity : 30 - 75% HR, Atmospheric Pessure : 700 - 1,060hPa	
Transportation and Storage Environment	Temperature : -10 – 50 °C, Humidity : 10 - 85% RH, Atmospheric Pessure : 500 - 1,060hPa	

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Elly-Beinhorn-Strasse 8, 65760 Eschborn, Germany

Visit our
website

The EU directive 93/42/EEC was applied in the design
and production of this medical device.

Specifications are subject to change without notice.

2021-08-XX CACB09XX XXN


Maintenance Guide

*for Air Turbine, Contra Angle
& Straight Handpiece*

Please read this Operation Manual carefully before use, and file for future reference.

OM-Z0135E 002

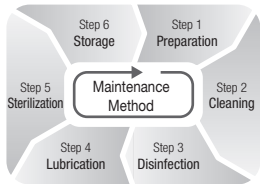This symbol on your NSK product means that the product can be cleaned and disinfected with a Thermo-disinfector.

This symbol on your NSK product means that the product can be autoclavable up to Max. 135°C.

1. Preparation

Remove Contra Angle/Turbine from the motor/coupling and detach the bur from the chuck.

2. Cleaning

Manual : Clean the external surface of handpiece with running water (<38°C, demineralised water recommended).

Automated : NSK handpieces with this symbol are the thermo-disinfector-ready and may be washed and disinfected by using a thermo-disinfector. When using a Thermo-disinfector, be sure to follow its manufacturer's instructions.

3. Disinfection

Manual : Wipe the external surface of handpiece carefully with cleaning or disinfectant solution.

Automated : 

4. Lubrication


Lubricate handpiece by using NSK PANA SPRAY Plus / PANA SPRAY. To prevent mist spray from escaping into the environment, use absorbent cloth. Expel excess lubrication.

Manual :

- 1) Push or screw nozzle. Shake the spray can 3-4 times.
- 2) Insert the spray nozzle in the rear of the handpiece and spray for approximately 2-3 seconds until the oil comes out of the handpiece head / nose.

Automated :

Clean and lubricate handpieces with NSK automatic handpiece cleaning and lubrication system. Be sure to use a right adaptor for each handpiece. Refer to the system instructions.

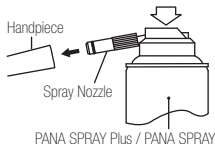⚠ CAUTION

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out by the spray pressure when lubricating.
- Keep the NSK PANA SPRAY Plus / PANA SPRAY can upright.
- Use of lubricants other than NSK's PANA SPRAY Plus / PANA SPRAY can cause excessive heat generation and possible handpiece damage.

5. Sterilization

Autoclave sterilization is recommended.

Sterilization is required first time you use and after each patient as noted below.

■ Autoclave Procedure :

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable up to a max. 135°C.
ex.) Autoclave for 20 min. at 121°C, or 15 min. at 132°C.

* Sterilization at 121°C for more than 15 minutes is recommended by EN13060 or EN ISO17665-1.

CAUTION of Sterilization

- Always place the handpiece in the center or upper shelf of the chamber, as the local temperature at the bottom of chamber could rise beyond the set value.
- Do not autoclave the handpiece even when it is bagged, together with other instruments, from which chemicals are not sufficiently cleaned. This is to avoid discoloration and/or damage the plating of the handpiece/motor.
- Do not heat or cool the handpiece quickly. Rapid change in temperature could break the glass rod and give abnormal strain to other metals. (Optic handpiece/motor)
- We do not recommend you to sterilize on plasma sterilization or EOG sterilization.

6. Storage

Immediately after a sterilization cycle, remove handpiece from the autoclave. Keep them in a dust-free sterilized storage or bring them to a treatment room for use.

CAUTION

- The handpiece is kept in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. Moreover, air does not include a dust, salt and sulfur.

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NSK Europe GmbH